

ActivNeck™ is comfortable to wear. It comprises a hard plate to support the neck, breathable cotton material against the skin to absorb moisture, and soft cushioning materials for comfort.



ActivNeck™ is suitable for stabilising the neck, mild sprain of the cervical vertebrae, whiplash, lesion of the cervical vertebrae, post-surgery recovery and neck disc injury.

How to wear:

- 1 Unfasten and open the neck brace**




If there are any loose fibres on the surface these pose no health risk. Just wipe off.

- 3 Close the velcro fastener**



Adjust to fit and close the velcro fastener.

- 2 Place the brace around your neck**

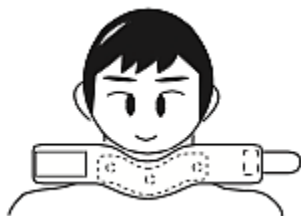

Place the neck brace around your neck making sure the plastic plate is in middle facing forward.

- 4 Finish**



Check that the neck brace is not too tight or too loose to wear. Make sure you feel comfortable support.

Caution

- When wearing for the first time, try this product for 1 to 2 hours
- Do not wear while sleeping
- Do not use this product other than explained in these user instructions as this can lead to damages or injuries
- Make sure the fastener does not stick to clothes as this may cause damage to clothes
- This product contains chemical fibres. If the product irritates the skin stop wearing the product immediately and consult your doctor

Washing:

- Hand wash with cold water
- No whitening or bleaching ingredients
- Air dry with good ventilation away from direct sunlight
- This product may lose colour because of water, sweat or friction
- Do not tumble dry

Storage:

- Store in a dry place with good ventilation
- Do not expose to direct sunlight or high temperature

Disposal:


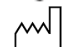
- Please conform with local guidelines for textile product disposal

ActivNeck™ has a strong internal support plate and soft breathable padding. The outside layer is made from moisture absorbing cotton. The supporter can be easily applied and removed with its velcro fastener.



Colour: Cream
One size fits all
Unisex

Handmade in Japan by:

 Daiya Industry Company, Ltd.
 1117-1 Koshinden, Minami-Ku Okayama 701-0203 Japan

Distributed in UK, EU and Australia by:

 **Gower**
Health & Fitness
Solutions

271 High Street, Berkhamsted HP4 1AA
United Kingdom
info@activsupports.com